REF	Description	Size	Unit
914671	Gel Wave - Blue & Yellow	13.5" x 21"	Each
914673	Gel Wave - Yellow & Green	13.5" x 21"	Each (as shown)

### Purpose

These trays incorporate multiple tactile sensations including 4 different fabrics and gel. They provide tactile and visual stimulation that addresses boredom, improvement of eyehand coordination, reduction of anxiety and finger strength and dexterity.

### Easy to Use

Place tray on a table or convenient surface and instruct the child to touch the surfaces, move the marbles and ask for feedback regarding sensations. Also encourage the child to remove and replace the four fabric shapes.

### Easy Care

The vinyl component is heat sealed and can be cleaned with a mild detergent. The fabric components can be cleaned with a mild detergent or can be laundered and dried at temperatures up to 180°F (82°C). Although the gel is non-toxic, medical care should be consulted if the gel is ingested. If a tear or rip is observed in the gel or fabric components the units should be discarded. Neither the gel or fabric components should be eaten or placed in the mouth. These products must always be used under the direct supervision of a care giver.

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