
REF	Description	Size	Units
909220	Stop Strip w/Stop Sign	12" x 50"	Each
909221	Stop Strip - Yellow (no stop sign)	12" x 50"	Each
909217	Stop Strip w/alarm	12" x 50"	Each

### Purpose

These brightly colored strips help to prevent accidents by directing residents away from potential hazard areas. Promotes resident privacy. Placed across a room doorway, this brightly colored strip often stops the cognitively impaired wanderer from entering another resident's room. It also acts as a reminder for the resident inside a room not leave without first notifying a staff member. Secured with VelcroR, the Stop-Strip is not a physical barrier, so it may be used as part of a restraint reduction program.

### Easy to Use

Peel the backing off the two Velcro® pressure-sensitive strips and attach them to the left and right sides of the door frame or to the wall on either side of the door frame. These strips should be positioned at midheight on the door frame or on the wall next to the door frame. (See illustration at right.) **IMPORTANT:** Attach the pressure sensitive strips to the corridor side of the door frame or wall so that the Stop-Strip is outside the room. (See illustration above.)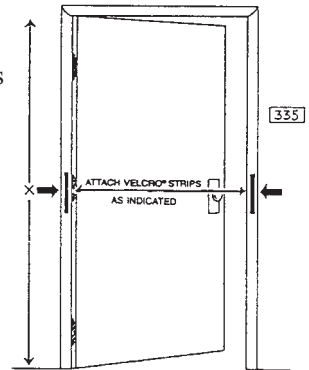

Do not place the Stop-Strip across fire and emergency exit doors.

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